0120

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
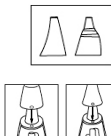Manual Instruction
NASAL ASPIRATOR
Model 1722

Thank you for purchasing a Momert product. We hope you will be satisfied with our product through its service life.

Please study the entire Operating Manual carefully before you start using the product. Keep the manual in a safe place for future reference. Make sure other people using the product are familiar with these instructions.


IMPORTANT SAFETY INSTRUCTIONS:

- The product can only be used with supervision for handicapped, reduced mental capability or unskilled people (children also). Don't let children play with it.
 - This device is to clean stuffy noses, do not use it in baby's mouth or ears.
 - Always clean and dry thoroughly before each use and after each use.
 - This is not a waterproof device, please clean it referring to the section of "Clean Instructions"
 - Do not insert the silicone tip into the nostril too deep.
 - Do not use it on the baby for a long time. A few seconds can be enough.
 - So not use this device if there is any wound, or symptom of inflammation or bleeding at the nose.
 - In case of infection, fever, nasal irritation or hemorrhage, please stop using this device and consult your physician.
 - This device is restricted for personal use.
 - The silicone tip is the applied part.
 - Duty cycle: 30 seconds ON / 2 minutes OFF
 - Replace batteries when the suction power becomes weak during use.
 - **Batteries are not included in warranty.**
- FAILURE TO FOLLOW THE MANUFACTURER'S INSTRUCTIONS MAY LEAD TO REFUSAL OF WARRANTY REPAIR!**

CONTENTS:


- 1 Main unit
- 2 sizes of silicone tips
- 1 instruction manual
- 1 cleaning tube
- 2 spare silicone rings
- 2 x 1,5V AA alkaline batteries (included)

PARTS OF DEVICE: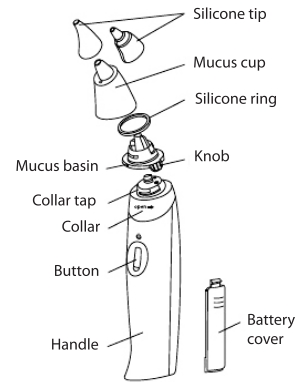


INSTRUCTIONS FOR USE:

1. Remove the battery cover.
2. Insert 2x1,5V AA alkaline batteries.
3. Place the battery cover back.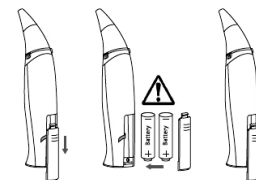

4. Use the suitable silicone tip based on the size of user's nostrils.
5. Be sure to place the silicone tip securely on the mucus cup.
6. Gently insert the silicone tip into the nostril not too deep.
7. Press the button for a few seconds to collect mucus.

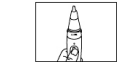
CLEANING INSTRUCTIONS:

1. Remove the silicone tip.
2. Use the finger and thumb to hold the 2 knobs of mucus basin from the main unit by turning it counterclockwise as "open" on the collar and then pull up.
3. Hold the mucus cup upside down. Use the finger and the thumb to hold the 2 knobs of mucus basin and turn it clockwise as "open" on the mucus basin.
4. Remove the silicone ring from the mucus basin.
5. Wash all 4 parts in warm soapy water and then clean them with stream or boiling water.



6. Assemble all the parts as shown below: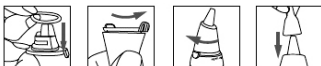

7. Be sure all the parts fit tightly together and line up the indication marks.

IF YOU FIND MUCUS OVER THE COLLAR TAP, PLEASE FOLLOW BELOW INSTRUCTIONS:

1. Please use attached cleaning tube. Put the cleaning tube to the suction hole.
2. Put the cleaning tube in the water and press the button for 30 seconds. Then lift the cleaning tube from the water and press the button again to pump out water inside till it is completely dry. Remove the cleaning tube and keep it for future use.

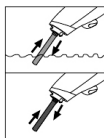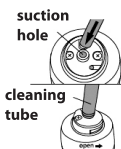**REPAIR AND MAINTENANCE:**

Any extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

ENVIRONMENTAL PROTECTION:

- Packaging materials and obsolete appliances should be recycled.
- The transport box may be disposed of as sorted waste.
- Polyethylene bags shall be handed over for recycling.

Appliance recycling at the end of its service life:

A symbol on the product or its packaging indicates that this product should not go into household waste. It must be taken to the collection point of an electric and equipment recycling facility. By making sure this product is disposed of properly, you will help prevent the negative effects on the environment and human health that would otherwise result from inappropriate disposal of this product. You can learn more about recycling this product from your local authorities, a household waste disposal service or in the shop where you bought this product.

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