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#### Modell 1758

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**Operation manual**  
**Digital thermometer**  
**Model 1758**

Thank you for purchasing a Momert product. We hope you will be satisfied with our product throughout its service life.

Please study the entire Operating Manual carefully before you start using the product. Keep the manual in a safe place for future reference. Make sure other people using the product are familiar with these instructions.

**Specification**

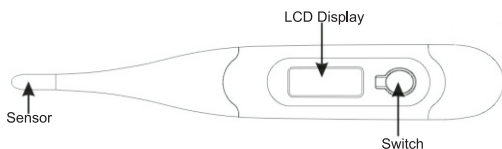
Measuring Range: 32.0°C ~ 42.9°C (90°F ~ 109.9°F)

Accuracy: ±0.1°C (±0.2°F) for 35.0 ~ 42.0°C (95°F ~ 107.6°F) & ±0.2°C (±0.4°F) for remaining range at an ambient of 22°C (71.6°F)

Battery: LR41 1.5V

Battery life: About 2 years with average use

Weight: About 12.5g

**How to use**

Press power switch, then screen displays "100.0", which indicates normal operation and sufficient power. Then it displays last measurement memory and starts a self-test routine. After self-test, the screen shows 37.0°C (98.6°F) if correct operation.

Release the button, then displays "Lo°C" or "Lo°F" and ready to measure. If temperature remains below 32.0°C (90°F), the symbol "Lo°C" or "Lo°F" will flash for about 60 seconds. Then it will auto-off for saving power.

During measurement, the symbol "Lo°C" or "Lo°F" will flash while the temperature is climbing.

-On getting a maximum value which doesn't change or the change is less than 0.1°C (0.2°F) within 16 seconds. The symbol "Lo°C" or "Lo°F" will cease flashing and the buzzer alarms for about 10 seconds

a) Normal alarm: Sound of "bi...bi...bi...bi..." means normal temperature that below 37.8°C (100°F).

b) Fever alarm: Sound of "bi,bi,bi,bi" means fever that the temperature is above 37.8°C (100°F).

LCD will display "Hi°C" or "Hi°F" if the measured temperature is above 43.0°C (109.9°F).

Turn off the unit by just slightly pressing power switch, or leave it for 10 minutes, it will auto-off for saving power.

Exchange function: If the power switch was pushed more than 2 sec after power-on, the unit "°C" or "°F" will be exchanged.

Children should be under close supervision at all times while using this product. After monitoring the temperature, the unit should be removed from the child immediately.

**Measurement:**

You are advised to measure your temperature at specific time in morning, if it is earlier or later, the temperature would be lower or higher.

Used in mouth: Please close your mouth for 1 minute for stability of temperature, breath calmly through nose. Respiratory air would affect the accuracy of your mouth. Normal temperature: About 37.0°C (98.6°F)

Used in anus of rectum: This is the way that can get the most steady temperature and suits for both infants and children. The sensor shall be inserted for 2~3 cm. Vaseline can be used for easier inserting. Normal temperature: About 37.6°C (99.7°F)

Used under armpit: Clean the thermometer with a dry cloth and put it under armpit. Press and fix it with the arm. The temperature value of this measurement is not so accurate that not recommended for accurate measurement Normal temperature: About 36.7°C (98.1°F)

Notes: The normal temperatures are different from the different human bodies and measuring time.

Sterilization is required before every measurement.

Measurement is inaccurate when the battery is low-power

**Replace Battery**

1) Every 24 months or when the display starts to be indistinct or when "Lo" appears on LCD, the battery must be replaced at once for accurate temperature.

2) To replace the battery, open the cover of the end part and take out the old battery with tools.

3) When replace with a new battery, make sure to use the new LR41, then re-insert the cover. Pull out the end cap. Slide the battery case out from the unit a little bit. Use a sharp pencil to take out the battery and insert a new battery. Make sure "negative" side is upward.

Remove battery from the unit when not in use for long time.

Be careful, keep the item away from children, because swallowing the battery or the cap can cause drowning.

Do not place battery near heat or it can explode!

Push the battery case into the unit and cover the end cap.

Battery is not included in warranty.

**Cleaning and maintenance:**

The thermometer is waterproof and can be cleaned by water or sterilization lotion with soft cloth. Sterilization lotion below is recommended: Name Density Medical alcohol 95%

Notes: Other lotion, density or cleaning method may damage the thermometer. Don't let the unit sit in alcohol and don't put it under hot water (higher temperature than 55°C), don't use ultrasonic cleaning.

The product doesn't need any calibration.

**Remarks:**

1) It is suitable for measuring temperature of human body and necessary of families.

2) It is designed and manufactured complying with relevant requirements in ISO 13485:2012/AC:2012; ISO 10993-1; ROHS: 2011/65/EU.

3) The device complies with the EMC EN 60601-1-2:2007 / AC:2010 as well as the medical 93/42 / EEC standards.

4) According to the degree of protection against electric shock, it is classified to BF, in accordance with EN60601-1-2.

5) Avoid falling, crash or knock.

6) Never put it in boiled water for sterilizing.

7) Never touch any chemical solution and keep away from straight sunlight or heat source as well as strong electromagnetic source to avoid interference.

8) It shall be operated in an environment at the temperature of 5~40°C (41~104°F) and the humidity of 5~95% while stored and transported in an environment at the temperature of -20~55°C (-4~131°F) and humidity of 5~95%

9) Don't remove any parts except replacing battery.

10) Please replace battery before claim for replacing a new product.

11) The scrap of product and battery are to be disposed according to relevant national regulations.

12) Caution! This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.

**REPAIR AND MAINTENANCE**

Any extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

**ENVIRONMENTAL PROTECTION:**

• Packaging materials and obsolete appliances should be recycled.

• The transport box may be disposed of as sorted waste.

• Polyethylene bags shall be handed over for recycling.

**Appliance recycling at the end of its service life:**

A symbol on the product or its packaging indicates that this product should not go into household waste. It must be taken to the collection point of an electric and equipment recycling facility. By making sure this product is disposed of properly, you will help prevent the negative effects on the environment and human health that would otherwise result from inappropriate disposal of this product. You can learn more about recycling this product from your local authorities, a household waste disposal service or in the shop where you bought this product.

  
.....  
Tamás Baritsa  
Chief Commercial Officer

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