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##### A Momert Zrt.,

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Tel.: + 36 / 25 / 555-100, Fax: + 36 / 25 / 555-199

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**Declare that product: Name:** Momert; **Model:** 3112; **Description:** Blood pressure monitor

##### Is in conformity with the following standards:

This Blood Pressure Monitor complies with the European regulations and bears the CE mark "CE 0197". This blood pressure monitor also complies with mainly following standards (included but not limited):

Safety standard:

EN 60601-1 Medical electrical equipment part 1: General requirements for safety

EMC standard:

EN 60601-1-2 Medical electrical equipment part 1-2: General requirements for safety- Collateral standard: Electromagnetic compatibility- Requirements and tests Performance standards:

EN 1060-1 Non-invasive sphygmomanometers - General requirements

EN 1060-3 Non-invasive sphygmomanometers - Supplementary requirements for electromechanical blood pressure measuring systems.

EN 1060-4 Non-invasive sphygmomanometers - Test procedures to determine the overall system accuracy of automated non-invasive sphygmomanometers.

Dunaújváros, 24. 11. 2015.

Momert Co. Ltd.

Tamás Baritsa  
Chief Commercial Officer

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**GB**  
INSTRUCTION MANUAL  
DIGITAL BLOOD PRESSURE MONITOR  
MODEL 3112

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## SPECIFICATIONS:

Product Description	Arm Type Fully Automatic Digital Blood Pressure Monitor	
Model	3112	
Display	LCD Digital Display	Size: 62,7 mm x 46,4 mm
Measurement Method	Oscillometric Method	
Measurement Range	Pressure	0mmHg~300mmHg
	Pulse	30 to 180 Beats/Minute
Measurement Accuracy	Pressure	±3mmHg
	Pulse	±5%
Pressurization	Automatic Pressurization	
Memory	120 Memory	
Function	Low Battery Detection	
	Automatic Power-off	
Power Source	4 AA batteries or Medical AC Adapter ( DC 6.0V, 600mA ) ( Recommended, not provided )	
Battery Life	Approximately 2 months at 3 tests per day	
Unit Weight	Approx.360g (12.36 oz.) (excluding battery)	
Unit Dimensions	Approx.134 x 99 x 66mm (5.36" x 3.96" x 2.64") (L x W x H)	
Cuff Circumference	22.0 cm~36.0 cm (8.66" ~ 15")	
Working Noise	<56.5dB (air pump)	
Operating Environment	Temperature	10 °C~40 °C (50 °F~104°F)
	Humidity	15-90% RH
	Pressure	Atmospheric Pressure
Storage Environment	Temperature	-20 °C~55 °C (-4 °F~131 °F)
	Humidity	15-90% RH
Classification	Internal Powered Equipment Type B 	

**BLOOD PRESSURE Q & A:**

**Q:** What is the difference between measuring blood pressure at home or at a professional healthcare clinic?

**A:** Blood pressure readings taken at home are now seen to give a more accurate account as they better reflect your daily life. Readings can be elevated when taken in a clinical or medical environment. This is known as White Coat Hypertension and may be caused by feeling anxious or nervous.

**NOTE:** Abnormal test results may be caused by:

**1. Improper cuff placement**

Make sure cuff is snug-not too tight or too loose. Make sure bottom of the cuff is approximately 1-2cm (1/2") above the elbow joint.

**2. Improper body position**

Make sure to keep your body in an upright position.

**3. Feeling anxious or nervous**

Take 2-3 deep breaths, wait a few minutes and resume testing.

**Q:** What causes different readings?

**A:** Blood pressure varies throughout the course of a day. Many factors including diet, stress, cuff placement, etc. may affect an individual's blood pressure.

**Q:** Should I apply the cuff to the left or right arm? What is the difference?

**A:** Either arm can be used when testing, however, when comparing results, the same arm should be used. Testing on your left arm may provide more accurate results as it is located closer to your heart.

**Q:** What is the best time of day for testing?

**A:** Morning time or any time you feel relaxed and stress free.

**MAINTENANCE:**

1. Avoid dropping, slamming, or throwing the unit.

2. Avoid extreme temperatures. Do not use outdoors.

3. When cleaning the unit, use a soft fabric and lightly wipe with mild detergent. Use a damp cloth to remove dirt and excess detergent.

4. Cuff Cleaning: Do not soak cuff in water! Apply a small amount of rubbing alcohol to a soft cloth

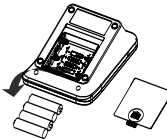
to clean cuff's surface. Use a damp cloth (water-based) to wipe clean. Allow cuff to dry naturally at room temperature.

5. Do not use petrol, thinners or similar solvents.

6. Remove batteries when not in operation for an extended period of time.

7. Do not disassemble product.

8. It is recommended the performance should be checked every 2 years.

**CORRECT DISPOSAL OF THIS PRODUCT**

(Waste Electrical & Electronic Equipment)

This marking shown on the product indicates that it should not be disposed with other household waste at the end of its life. To prevent potential harm to the environment or to human health, please separate this product from other types of wastes and recycle it responsibly. When disposing this type of product, contact the retailer where product was purchased or contact your local government office for details regarding how this item can be disposed in an environmentally safe recycling center. Business users should contact their supplier and check the terms and conditions of the purchasing agreement. This product should not be mixed with other commercial wastes for disposal. This product is free of hazardous materials.

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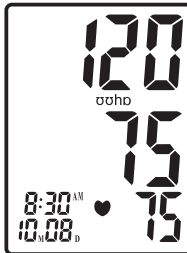
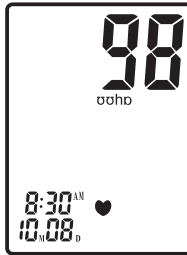
measurement memory as NO:01 reading. Older test result in memory can be viewed by pressing the "MEM" button.

#### Memory Deletion

While in memory check mode. Press "START/STOP" and hold on for about 3 seconds to delete all history results and the LCD screen display "---" with beep sounds. Then press "START/STOP" button to turn off the unit.

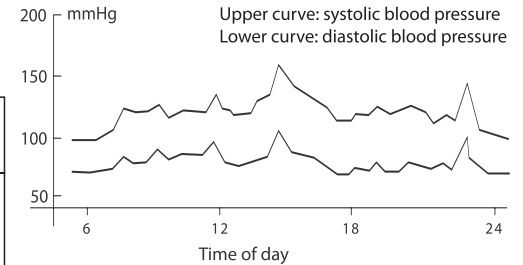
#### TROUBLESHOOTING:

Problem	Possible Cause	Solution
Blood pressure results are not within typical range.	Cuff is too tight or not properly positioned on the arm.	Firmly reposition cuff approximately 1-2cm (0.4"-0.8") above the elbow joint.
	Inaccurate test results due to body movement or monitor movement.	Sit in a relaxed position with arm placed near heart. Avoid speaking or moving body parts while testing. Make sure the monitor unit is placed in a stationary position throughout the testing period.
"Err" displayed	Cuff fails to inflate properly.	Make sure hose is properly fastened to cuff and monitor unit.
	Improper operation.	Read user manual carefully and re-test properly.
	Pressurization is over 300 mmHg	Read user manual carefully and re-test properly.

#### BLOOD PRESSURE INFORMATION:

**Blood Pressure:** Blood pressure is the force of blood pushing against the walls of arteries. It is typically measured in millimeters of mercury (mmHg.) Systolic blood pressure is the maximum force exerted against blood vessel walls each time the heart beats. Diastolic blood pressure is the force exerted on blood vessels when the heart is resting between beats. An individual's blood pressure frequently changes throughout the course of a day.


Excitement and tension can cause blood pressure to rise, while drinking alcohol and bathing can lower blood pressure. Certain hormones like adrenaline (which your body releases under stress) can cause blood vessels to constrict, leading to a rise in blood pressure. If these measuring numbers become too high, it means the heart is working harder than it should.


**EXAMPLE:** Fluctuation within a day (male, 35 years old).


**Health Reminder:** Hypertension is a dangerous disease that can affect the quality of life. It can lead to a host of problems including heart failure, kidney failure, and cerebral hemorrhaging. By maintaining a healthy lifestyle and visiting your physician on a regular basis, hypertension and relative diseases are much easier to control when diagnosed in their early stages.

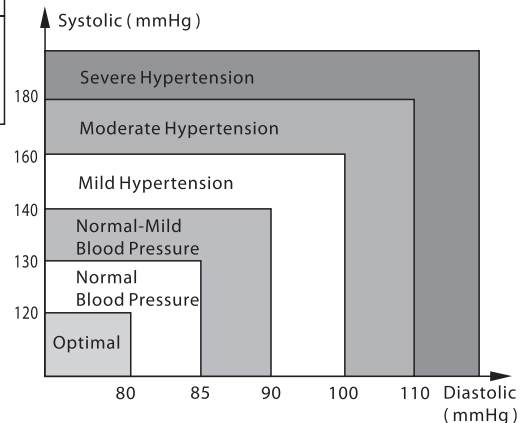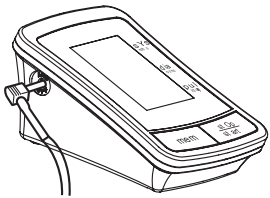**NOTE:** Do not be alarmed if an abnormal reading occurs. A better indication of an individual's blood pressure occurs after 2-3 readings are taken at the same time each day over an extended period of time. Consult your physician if test results remain abnormal.

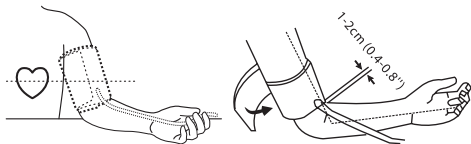3. Fasten cuff about 1-2cm (0.4-0.8") above the elbow joint. For best results apply cuff to bare arm and keep level with heart while testing.

**NOTE:** Do not insert air plug into opening located on up side of monitor unit. This opening is designed for an optional power supply only.

#### Time/Date Setting

With power off, press "START/STOP" button about 3 seconds to set the Time/Date mode. Set month first by adjusting the "MEM" button, Press "STOP/START" button to set the day, hour, and minute in same way. No matter in which setting mode, press "START/STOP" button to turn the unit off. All information will be saved.

**Note:** if unit is left on and not in use for 3 minutes, it will automatically save all information and shut off.

#### TESTING

1. Power On: Press and hold "Start/Stop" button until a beep sounds. The LCD screen will appear for one second as unit performs a quick diagnosis. A long beep sound indicates device is ready for testing.

**NOTE:** Unit will not function if residual air from previous testing is present in cuff. The LCD will flash "↓" until pressure is stabilized.

2. Pressurization: Press "START" button, the unit will start to inflate automatically. Initial pressure is first pumped to 190 mmHg. If the current user's systolic blood pressure is over 190mmHg, the unit will automatically re-inflate to the proper level.

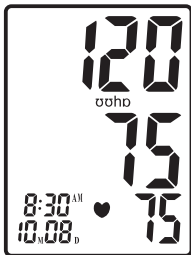
**NOTE:** Pressurization will gradually subside and ultimately stop when cuff is not properly applied to the arm. If this occurs, reposition the cuff and continue by pressing the "START" button.

3. Testing: After cuff inflation, air will slowly subside as indicated by the corresponding cuff pressure value. A flashing "♥" will appear simultaneously on screen signaling heart beat detection.

**NOTE:** Remain relaxed during testing. Avoid speaking or moving body parts.

4. Result Display: Three short beeps sound when testing is complete. The screen will display measurement between "mmHg" scale.

5. Power Off: The "START/STOP" button can be pressed to turn off the unit in any mode. The unit can turn off the power itself about 3 minutes if no operation in any mode.

**SAFETY PRECAUTION:** If pressure in arm cuff becomes too extreme while testing, press "Start/Stop" button to turn power off. The cuff pressure will rapidly dissipate once the unit is off.

#### Memory Check

With power off, press and hold "MEM" button to turn the unit on. The LCD will display the last

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## IMPORTANT TESTING GUIDELINES:

1. Avoid eating, exercising, and bathing for 30 minutes prior to testing.
2. Sit in a calm environment for at least 5 minutes prior to testing.
3. Do not stand while testing. Sit in a relaxed position while keeping your arm level with your heart.
4. Avoid speaking or moving body parts while testing.
5. While testing, avoid strong electromagnetic interference such as microwave ovens and cell phones.
6. Wait 3 minutes or longer before re-testing.
7. Try to measure your blood pressure at the same time each day for consistency.
8. Test comparisons should only be made when monitor is used on the same arm, in the same position, and at the same time of day.
9. This blood pressure monitor is not recommended for people with severe arrhythmia.
10. Do not use this blood pressure monitor if the device is damaged.

## QUICK START:

1. Install batteries. (See Figure A)
2. Insert cuff air plug into the left side of monitor unit. (See Figure B)

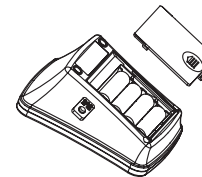
Figure A

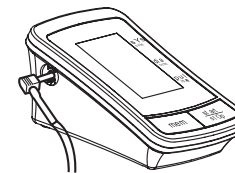
Figure B

3. Remove thick clothing from the arm area.
4. Rest for several minutes prior to testing. Sit down in a quiet place, preferably at a desk or table, with your arm resting on a firm surface and your feet flat on the floor. (See Figure C)

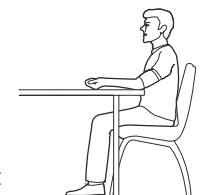
Figure C

5. Apply cuff to your left arm and keep level with your heart. Bottom of cuff should be placed approximately 1-2cm (0.4-0.8") above elbow joint. (See Figures D & E)
6. Press "START/STOP" Button to turn power on, and then press "START" button to start testing.

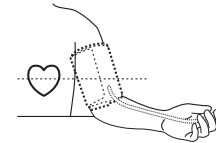
Figure D



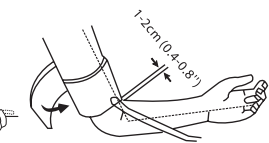
Figure E

## UNIT OPERATION:

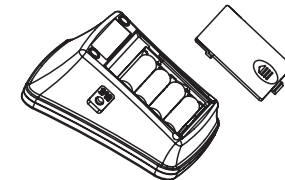**Battery Installation:** Slide battery cover off on the back side of the unit. Install 4 new AA alkaline batteries according to polarity. Close battery cover.

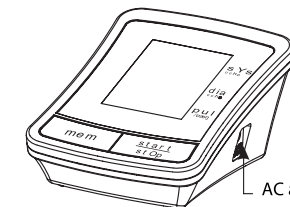AC Adapter jack is on the front side of the monitor. Medical AC adapter (DC 6.0 V, 600mA) can be used with the device (recommended, not provided). The adapter connect pin should be positive inside and negative outside with a 2.1mm coaxial joint. Do not use any other type of AC adaptor as it may harm the unit.


AC adapter jack

## Applying the Arm Cuff:

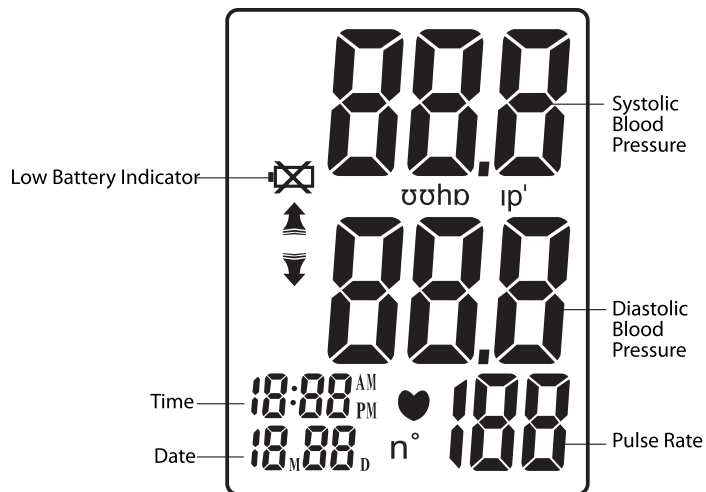
1. Firmly insert air plug into opening located on left side of monitor unit.
2. With sticky nylon section facing outward, insert end of cuff underneath metal ring of cuff.


## Display:


## Contents:

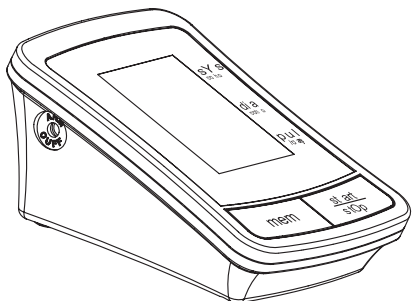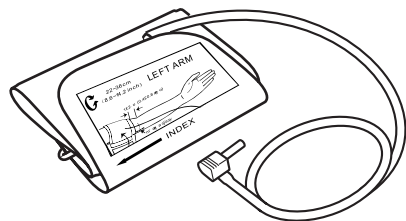
1. Monitor Unit

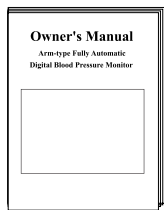3. Arm Cuff

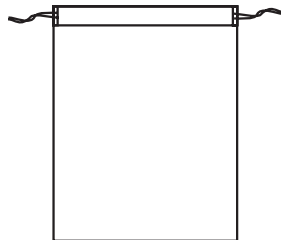2. Owner's Manual

4. Storage Bag

5. Medical AC Adapter (DC6.0 V, 600mA)  
(recommended, not provided)

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# INSTRUCTION MANUAL DIGITAL BLOOD PRESSURE MONITOR MODEL 3112

Thank you for buying a Momert product, and wish to use it for a long time with satisfaction. Before the first use of product please read this manual carefully, and keep it for later. Please be sure that other people using the product will also read the manual.

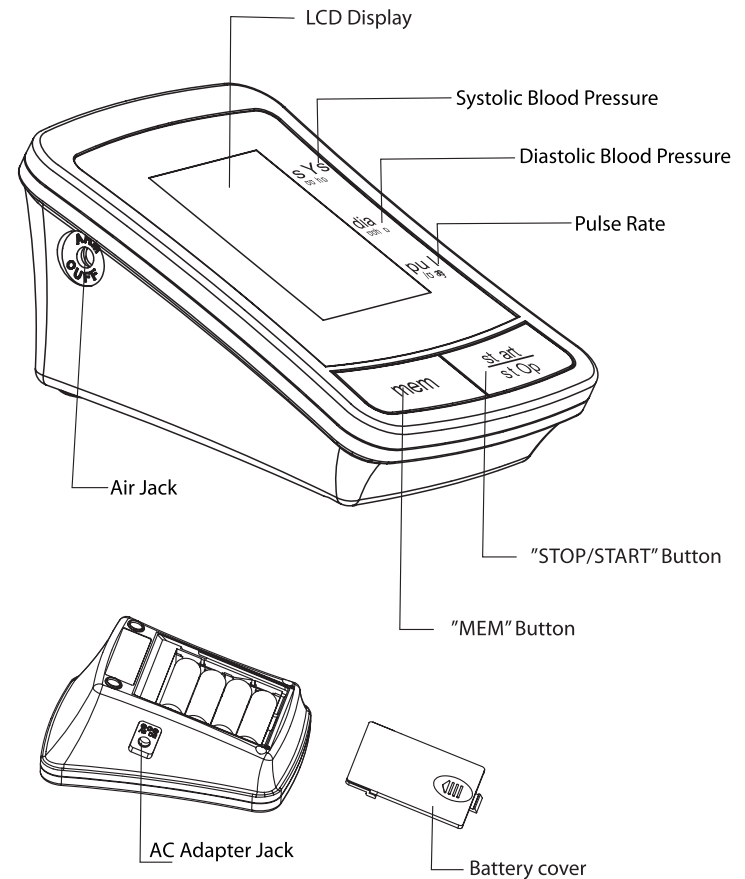
### CAUTION!

- This device is intended for adult use only.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- This device is intended for non-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on extremities other than the wrist or for functions other than obtaining a blood pressure measurement.
- Do not confuse self-monitoring with self-


diagnosis. This unit allows you to monitor your blood pressure. Do not begin or end medical treatment based solely physician for treatment advice.

- If you are taking medication, consult your physician to determine the most appropriate time to measure your blood pressure. Never change a prescribed medication without consulting your physician.
- This unit is not suitable for continuous monitoring during medical emergencies or operations.
- If the cuff pressure exceeds 40 kPa (300 mm Hg), the unit will automatically deflate. Should the cuff not deflate when pressures exceeds 40 kPa (300 mmHg), detach the cuff from the wrist and press the start/stop button to stop inflation.
- To avoid measurement errors, carefully read this manual before using the product.
- The equipment is not AP/PG equipment and not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- The operator shall not touch output of AC adaptor and the patient simultaneously.
- To avoid measurement errors, please avoid the condition of strong electromagnetic field radiated interference signal or electrical fast transient/burst signal when using the AC adaptor.
- The user must check that the equipment functions safely and see that it is in proper working condition before being used.

### Monitor unit:

### Arm cuff:
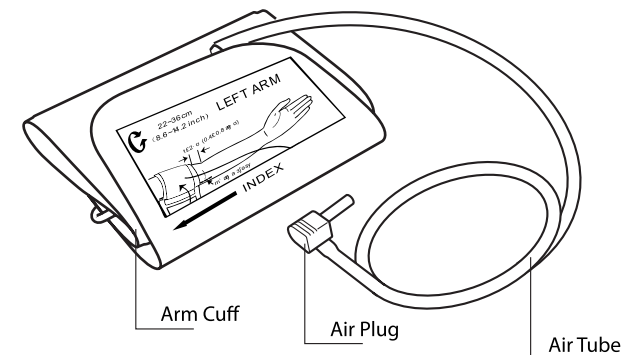
### WARNING SIGNS AND SYMBOLS USED IN THIS OWNER'S MANUAL

	Caution		Mandatory		Prohibited		Type B Equipment
--	---------	--	-----------	--	------------	--	------------------

#### BATTERY PRECAUTIONS

- Do not mix new and old batteries simultaneously.
- Replace batteries when Low Battery Indicator "" appears on screen.
- Be sure battery polarity is correct.
- Do not mix battery types. Long-life alkaline batteries are recommended.
- Remove batteries from device when not in operation for more than 3 months.
- Dispose batteries properly; observe local laws and regulations.

#### CAUTION

- Individuals with serious circulation problems may experience discomfort. Consult your physician prior to use.
- Contact your physician if test results regularly indicate abnormal readings. Do not attempt to self-treat these symptoms without consulting your physician first.
- Product is designed for its intended use only. Do not misuse in any way.
- Product is not intended for infants or individuals who cannot express their intentions.
- Do not disassemble or attempt to repair.
- Do not use cell phones near the unit as they may cause incorrect readings.
- Only use a recommended AC adaptor (see Page 27.) An unauthorized adaptor may cause fire and electric shock.