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**WYSWIETLACZ:**

**ZASILACZ:**

**OPERATING MANUAL  
ELECTRONIC COLUMN SCALE  
MODEL 5963**

Thank you for purchasing a Momert product. We hope you will be satisfied with our product throughout its service life. Please study the entire Operating Manual carefully before you start using the product. Keep the manual in a safe place for future reference. Make sure other people using the product are familiar with these instructions.

**SPECIFICATIONS:**

- Equipped with 4 high-precision strain gauge sensor system.
- With 1LCD display on the top of the column
- Buttons functioning with sound signal

Technical parameters	
Voltage	9V alkaline battery or mains adapter (included)
Max. measurable weight	200 kg
Weighing increments	100 g
Min. measurable weight	2 kg
Platform	White painted platform with safety and non-skidding mat
Weighing units	kg,oz,stlb
Working/Storage temperature	10-40°C/10-50°C

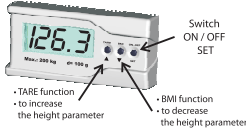
**IMPORTANT SAFETY INSTRUCTIONS:**

- Make sure you follow the marked polarity of battery when inserting and replacing it.
- If the appliance will be out of use for a longer time, remove the battery.
- Do not leave the unit unattended when in use.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- Use the product only for the intended purpose specified in the Operating Manual.
- Use the unit only on a stable horizontal surface.

- Before cleaning, assembly, or disassembly of the cover, switch the unit off and remove the battery!
- Do not use abrasive or chemically aggressive substances for cleaning the unit.
- Do not use the unit if it does not operate properly, or if it has been dropped, damaged or exposed to any liquids. Take the unit to an authorized service center for testing and repair.
- The unit is suitable for domestic use only. It is not designed for commercial operation.
- Keep the unit away from heat sources, such as radiators, ovens etc. Do not expose the unit to direct sunlight or moisture.
- Do not touch the unit with wet or damp hands.
- The product is a fragile measuring device. Prevent it from crashing or falling.
- Do not repair the appliance yourself. Contact an authorized service center.

**FAILURE TO FOLLOW THE MANUFACTURER'S INSTRUCTIONS MAY LEAD TO REFUSAL OF WARRANTY REPAIR!**

**DISPLAY:**

**ADAPTER TO THE MAINS:**

**USER GUIDE:**

1. Switched on the scale with the ON/OFF button. Push the button two times quickly, so the height parameter flashes on the display.
2. Press up or down arrow buttons to indicate the height.

- Then press SET to confirm the data. After this "0.0" is displayed, the scale is ready for measuring.
- Stand on the scale and wait until measuring is finished.
- Press the BMI button once, then the scale counts your BMI value.
- Stand on the measuring platform. After display shows the weight press the BMI/HOLD button. Then HOLD sign will appear on display.
- Press again the BMI/HOLD button. Then weight and BMI value will blink on display alternately.
- The body weight and the counted BMI value flash 5 times alternately, then the scale automatically switches off in 30 seconds.
- If you wish to turn off the scale before the 30 seconds, just press the ON-OFF button once. The height parameter will not erase.
- If the scale is only used for measuring weight, and you don't want to check the BMI value, then the scale can be turned off by pressing the ON/OFF button for 3 seconds.

#### BMI - Body Mass Index = Weight / Height<sup>2</sup>

underweight	normal	overweight	obese
< 18,5	18,5 - 24,9	25 - 29,9	≥ 30

#### USING TARE FUNCTION:

By pressing TARE button after weight measuring the weighted value will be erased, on display "0.0" will be shown, so the scale is ready for another measuring.

#### CHANGING UNITS:

During measuring, or when display shows "0 g", the unit (kg, oz, st, lb) can be changed with the slide switch at the back side of the display box.

#### MAINTENANCE:

- Please keep the scale on a clean and horizontal surface.
- Keep the scale away from extreme low and high temperature, and from damp places.
- Do not put weight too sudden on it or drop the scale, because it has got high precision gauge sensors in it.
- Do not overweight the scale.

- When cleaning the scale do not use abrasive cleaning agents as they may cause damage.
- If you do not use the scale often or when you use mains adapter, please remove the battery.

#### REPAIR AND MAINTENANCE:

Any extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

#### ENVIRONMENTAL PROTECTION:

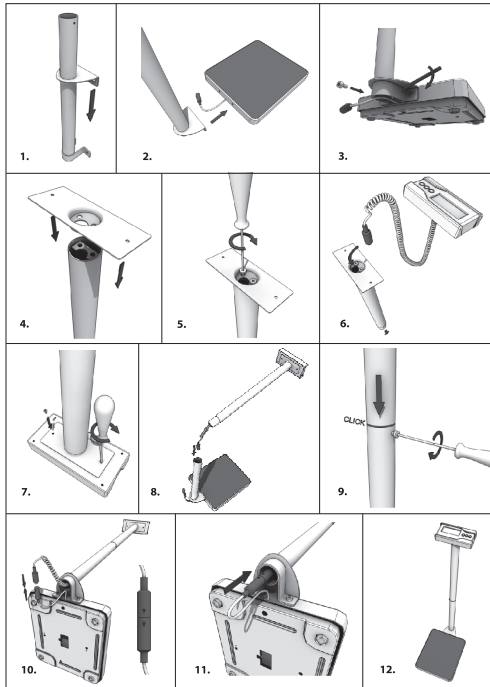
- Packaging materials and obsolete appliances should be recycled.
- The transport box may be disposed of as sorted waste.
- Polyethylene bags shall be handed over for recycling.

#### Appliance recycling at the end of its service life:

A symbol on the product or its packaging indicates that this product should not go into household waste. It must be taken to the collection point of an electric and equipment recycling facility. By making sure this product is disposed of properly, you will help prevent the negative effects on the environment and human health that would otherwise result from inappropriate disposal of this product. You can learn more about recycling this product from your local authorities, a household waste disposal service or in the shop where you bought this product.

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8

13

- Then press SET to confirm the data. After this "0.0" is displayed, the scale is ready for measuring.
- Stand on the scale and wait until measuring is finished.
- Press the BMI button once, then the scale counts your BMI value.
- Stand on the measuring platform. After display shows the weight press the BMI/HOLD button. Then HOLD sign will appear on display.
- Press again the BMI/HOLD button. Then weight and BMI value will blink on display alternately.
- The body weight and the counted BMI value flash 5 times alternately, then the scale automatically switches off in 30 seconds.
- If you wish to turn off the scale before the 30 seconds, just press the ON-OFF button once. The height parameter will not erase.
- If the scale is only used for measuring weight, and you don't want to check the BMI value, then the scale can be turned off by pressing the ON/OFF button for 3 seconds.

#### BMI - Body Mass Index = Weight / Height<sup>2</sup>

underweight	normal	overweight	obese
< 18,5	18,5 - 24,9	25 - 29,9	≥ 30

#### USING TARE FUNCTION:

By pressing TARE button after weight measuring the weighted value will be erased, on display "0.0" will be shown, so the scale is ready for another measuring.

#### CHANGING UNITS:

During measuring, or when display shows "0 g", the unit (kg, oz, st, lb) can be changed with the slide switch at the back side of the display box.

#### MAINTENANCE:

- Please keep the scale on a clean and horizontal surface.
- Keep the scale away from extreme low and high temperature, and from damp places.
- Do not put weight too sudden on it or drop the scale, because it has got high precision gauge sensors in it.
- Do not overweight the scale.

- When cleaning the scale do not use abrasive cleaning agents as they may cause damage.
- If you do not use the scale often or when you use mains adapter, please remove the battery.

#### REPAIR AND MAINTENANCE:

Any extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

#### ENVIRONMENTAL PROTECTION:

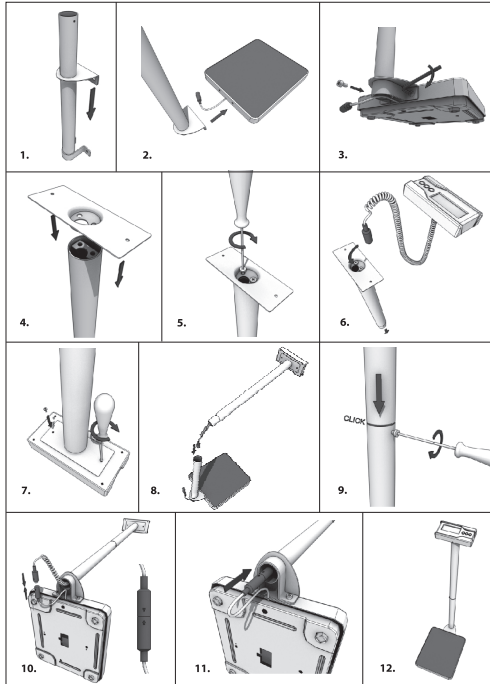
- Packaging materials and obsolete appliances should be recycled.
- The transport box may be disposed of as sorted waste.
- Polyethylene bags shall be handed over for recycling.

#### Appliance recycling at the end of its service life:

A symbol on the product or its packaging indicates that this product should not go into household waste. It must be taken to the collection point of an electric and equipment recycling facility. By making sure this product is disposed of properly, you will help prevent the negative effects on the environment and human health that would otherwise result from inappropriate disposal of this product. You can learn more about recycling this product from your local authorities, a household waste disposal service or in the shop where you bought this product.

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